


Healgen® Accurate Urine Drug Test Dip Card

16 Drugs in
Any
Combination

The Accurate Urine Drug Test Dip Card is a rapid qualitative immunoassay for the preliminary detection of up to sixteen (16) drugs or drug metabolites in urine with or without adulteration/specimen validity testing.

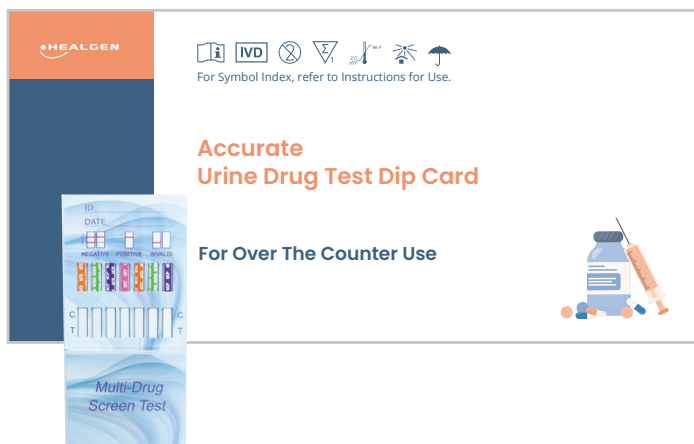

Kit Contents

*some components only available in the 25 test size

- Dip Card, individually sealed
- Specimen Validity/Adulteration Color Card (if applicable)
- Instructions for Use
- Procedure Card
- Disposable Gloves

Features & Benefits

Flexible Test Panels

Multiple drug configurations to suit different testing needs with or without adulteration/specimen validity tests (SVT).

Simple Collection and Testing Procedure

User-friendly process that can be performed without extensive training.

Easy Visual Interpretation

No instrumentation required.

Fast Results

Results in 5 minutes enable quick decision making. Convenient use in multiple settings.

Compact Design

Easy to store and transport.

510(k) Cleared & CLIA Waived


Use in multiple testing scenarios.

Cost-Effective

Affordable pricing without compromising quality.

Test Procedure

1. Remove the Test Dip Card and use within one hour.
2. Remove the cap from the Test Dip Card.
3. Dip the absorbent tip of the Test Dip Card into the urine sample for 10-15 seconds.
4. Ensure the urine level is below the maximum (MAX) line on the strip.
5. Replace the cap over the absorbent tip and lay the device flat. Read the results after 5 minutes.

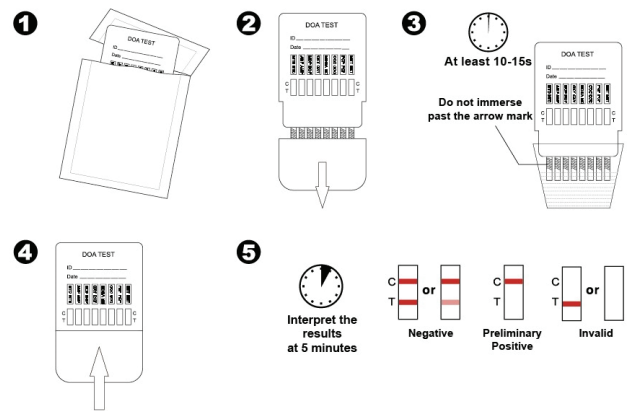

Test Menu

Test	Calibrator	Cut-off (ng/mL)
6-MAM	6-Monoacetylmorphine	10
AMP	d-Amphetamine	500 / 1000
BAR	Secobarbital	300
BUP	Buprenorphine	10
BZO	Oxazepam	300
COC	Benzoyllecgonine	150 / 300
EDDP	2-ethylidene-1,5-dimethyl-3, 3-diphenylpyrrolidine	300
MDMA	Methylenedioxymethamphetamine	500
MET	d-Methamphetamine	500 / 1000
MTD	Methadone	300
OPI	Morphine	300 / 2000
OXY	Oxycodone	100
PCP	Phencyclidine	25
PPX	d-Propoxyphene	300
TCA	Nortriptyline	1000
THC	11-nor- Δ^9 -THC-COOH	50

Ordering Information

Cleared for Over-the-Counter use

Description	Kit Size
Accurate Home Urine Drug Screen Dip Card	1/2/5 tests/kit
Accurate Urine Drug Test Dip Card	25 tests/kit