

Healgen® Rapid Check® COVID-19/Flu A&B Antigen Test Pro



For in vitro Diagnostic Use Only.
For Professional Use Only.
De Novo Authorized.

The **Healgen® Rapid Check® COVID-19/Flu A&B Antigen Test Pro** is a lateral flow immunochromatographic assay intended for the qualitative detection and differentiation of influenza A, and influenza B nucleoprotein antigens and SARS-CoV-2 nucleocapsid antigen directly in anterior nasal swab samples from individuals with signs and symptoms of respiratory tract infection.

Symptoms of respiratory infections due to SARS-CoV-2 and influenza can be similar. This test is for use by individuals aged 14 years or older testing themselves, or adults testing individuals aged 2 years or older.

Features & Benefits:

SARS-COV-2

Positive Percent Agreement



92.0%

Negative Percent Agreement



99.0%

Flu A

Positive Percent Agreement



92.5%


Negative Percent Agreement



99.9%

Flu B

Positive Percent Agreement

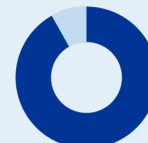

90.5%


Negative Percent Agreement



99.9%

- Excellent test performance (comparator RT-PCR)
- Rapid detection - results available in 15 minutes
- Simultaneously detects 3 viruses at once (SARS-CoV2, Flu A&B)
- External control availability - fits all QC testing scenarios
- Easy readability - two result windows 24 month expiration
- Store between 36°F - 86°F (2°C - 30°C)

Test Procedure:

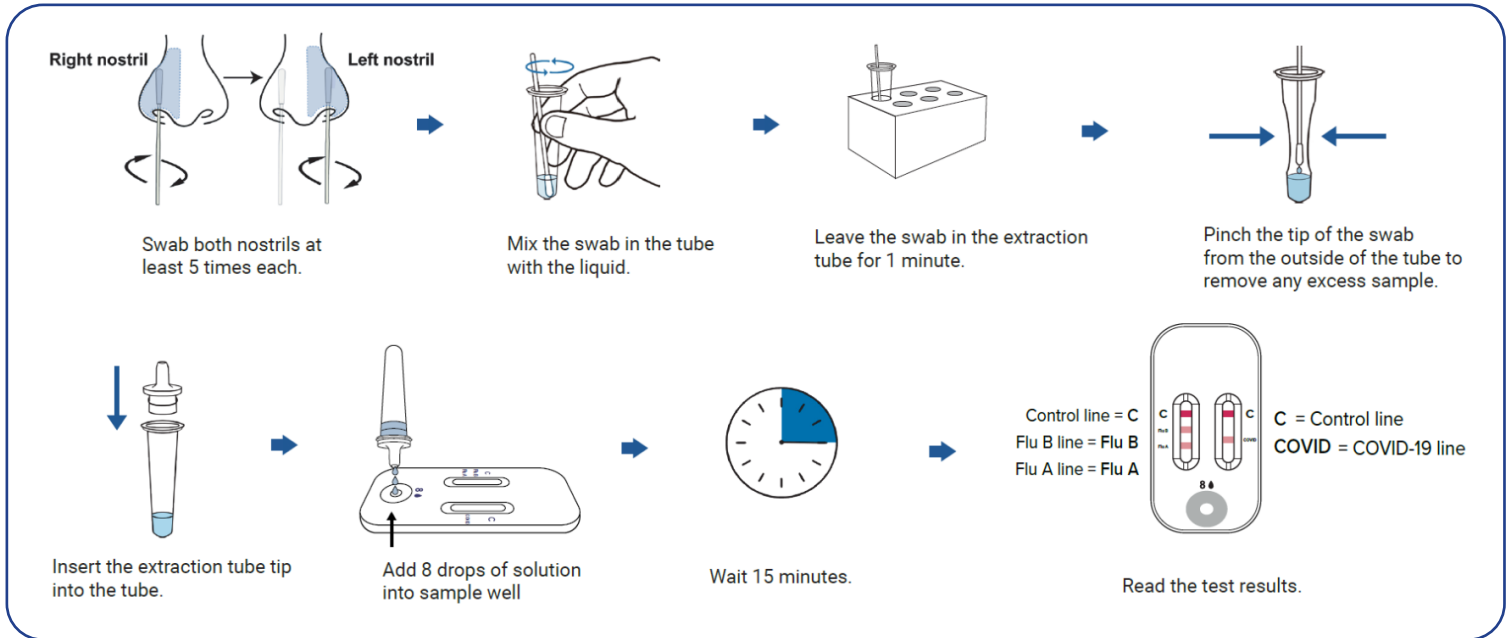

Kit Contents:

- 25 Sealed Test Cassettes
- 25 Sterile Nasal Swabs
- 25 Pre-filled Extraction Tubes
- 25 Extraction Tube Tips
- 2 Tube Holders
- 1 Instructions For Use (IFU)
- 1 Quick Reference Instructions (QRI)

Learn more about
this product online,
just scan the QR Code!



Ordering Information:

Product Name	Catalog No.	Kit Size	Regulatory Status
Healgen® Rapid Check® COVID-19/Flu A&B Antigen Test Pro	GCFC-525SKa-10T	10 Test/Kit	De Novo
	GCFC-525SKa-25T	25 Test/Kit	
External Controls: Healgen® Rapid Check® COVID-19/Flu A&B Antigen Test Pro	GCFC-PNK2	1 Positive & 1 Negative Control Swab	FDA Exempt
	GCFC-PNK20	10 Positive & 10 Negative Control Swabs	
	GCFC-PNK40	20 Positive & 20 Negative Control Swabs	

Healgen Scientific, LLC. 330 Barker Cypress Rd, Houston, TX 77094
 Toll-Free: +1 (866) 982-3818 (Mon - Fri 8:30 a.m. to 5 p.m. CT)
 Customer Service: customer.support@healgen.com
 Sales: sales@healgen.com
 Website: www.healgen.com

